

## HD75 – OTC Hearing Aid

\$280.00 - \$595.00

- **45 Day Money Back Guarantee!**
- Easy User Customization without a smartphone using Presets on board using buttons on the device
- Advanced Customization with Free [Customizer App](#) for Android & iPhone
- Size 312 non-rechargeable battery - Up to 3.5 days of use per battery
- Integrated Bluetooth for app use only, no streaming available
- Directional microphones for optimum performance in noisy conditions
- Environmental profile settings for best sound in different locations
- Flexible tube to accommodate most ear lengths from top of ear to canal opening
- Standard 1-year manufacturer's warranty included
- **Note:** Like all hearing aids, SWS products can fail when exposed to excessive moisture such as high humidity and sweating. Because moisture damage is considered normal wear, it is NOT covered by the manufacturer's warranty. To prolong the life of your devices, we suggest you remove them in high-moisture situations and consider investing in a hearing aid dehumidifier to regularly help dry them out.

Get the HD75 App here:



Option

Warranty  [Clear](#)

\$280.00

Add to cart

Description Shipping Information App Features

### Description

#### HD75 Specs

- Max Gain: 42dB
- Max Output: 117dB SPL
- [Complete Spec Sheet](#)
- Latency: Less than 3 msec

#### Digital Audio Processing

- 16 channel compressor
- Feedback cancellation



- 3 built-in amplification presets selectable by buttons on the device
- Volume control
- Further customization can be done with the free HD75 mobile app

#### Free HD75 Mobile App

- Android version 5.0 and newer
- iOS 9 and newer
  - iPhone 4S and newer
  - iPad 4th generation and newer
  - iPad Mini 2 and newer
  - iPad Air 2 and newer
  - iPad Pro and newer

**Fit:** Accommodates most ear lengths with flexible tube

**Bluetooth compatibility:** Bluetooth 5.2 for app use only, no streaming available

**Voice Prompts:** Environment Mode Name (Everyday, Restaurant, Entertainment), Volume Up, Volume Down, Connected, Connection Lost, Power Off

**Voice Prompt Languages:** English

**Battery type:** Size 312 non-rechargeable zinc air

**Battery life:** up to 3.5 days per battery

#### Box Contents

- HD75
- Size 312 battery pack (s)
- 5 different sized ear tips: x-small, small, medium (attached to HD75), large, x-large
- Ear tip cleaning tool
- Zippered carrying case
- [User Guide](#)

**WARNING: If you are younger than 18, do not use this.** You should go to a doctor, preferably an ear-nose-throat doctor (and ENT), because your condition needs specialized care. Over-the-counter hearing aids are only for users who are age 18 or older.

Orders are processed during regular business hours: Monday through Friday, 9:00 am to 5:00 pm CST. Orders with standard shipping typically take 2 business days to process for shipment. Expedited orders are typically processed the same day if received by 12:00 pm CST.

If you experience any problems with your Sound World Solutions product, visit our [Frequently Asked Questions](#) page. If you are unable to resolve the issue, please contact our customer support team between 9:00 am and 1:00 pm CST Monday through Friday toll-free in the U.S. at (855) 792-0117. Outside the US, call +1-847-939-6101 for customer service or technical support.

We accept MasterCard, Visa, American Express, Discover, and PayPal.


# HD75 Hearing Aid/Personal Sound Amplifier

## PRODUCT FEATURES



### Features:

- Color: Two-tone grey
- 16-Channel dynamic compression
- 3-Channel output compression/limiting
- 16-Channel noise reduction
- Feedback cancellation
- Directional microphones
- Volume, environment and preset controls

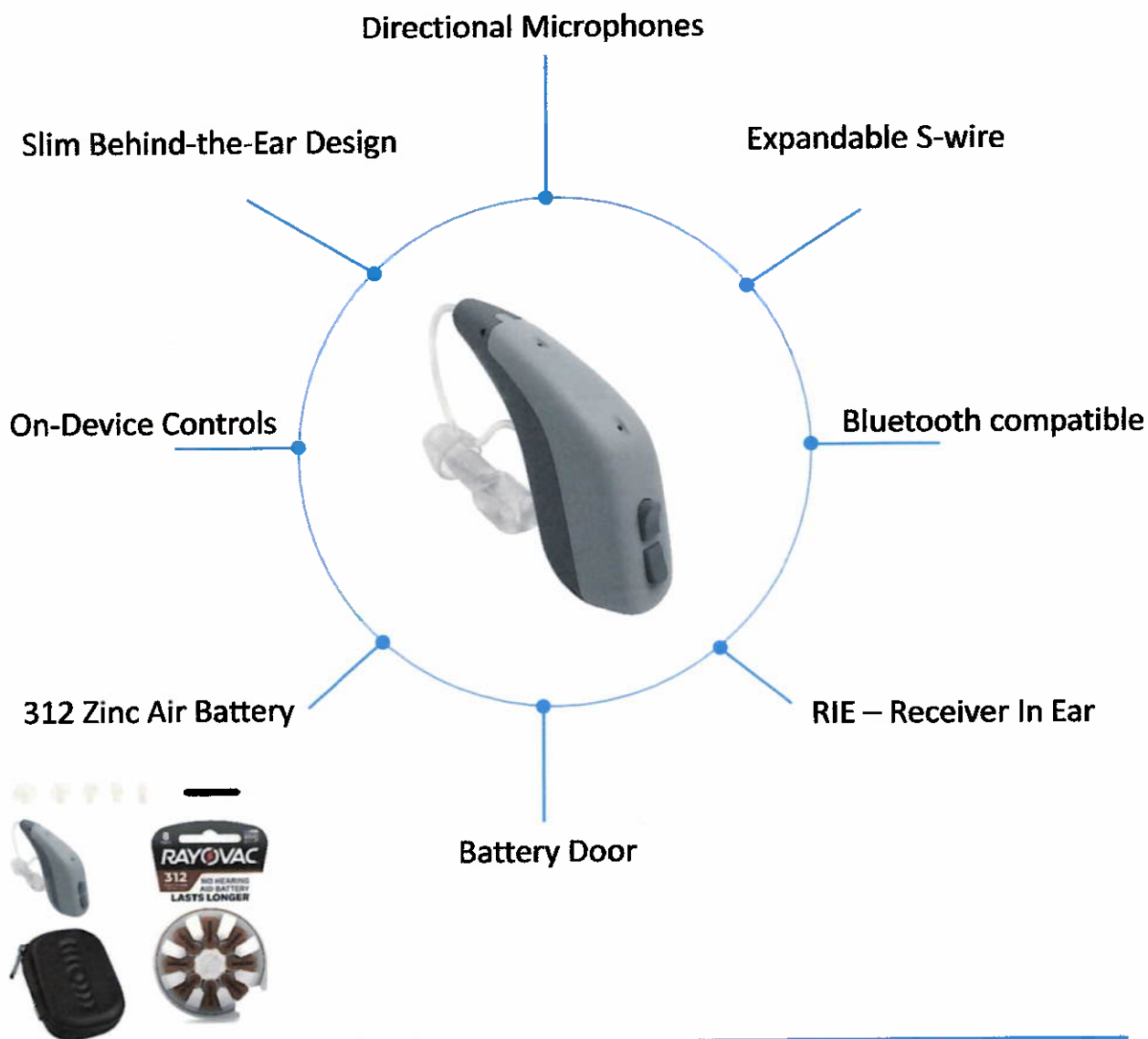


# HD75 Hearing Aid/Personal Sound Amplifier

## TECHNICAL SPECIFICATIONS



|                 |   |
|-----------------|---|
| Product Type    | Hearing Aid (HA), Personal Sound Amplifier (PSA)                          |
| Form Factor     | BTE/RIE   |
| Battery         | 312 Zinc Air Hearing Aid Battery  |
| Battery Life    | 60 Hours  |
| Programmability | Via Bluetooth connection to Smartphone, tablet; or via controls on device |
| Gain/Output     | 42 dB of gain and 117 dB SPL of maximum output                            |
| Latency         | Less than 3 msec  |
| Dimensions      | 13 mm wide x 31 mm high x 7.4 mm deep                                     |
| Availability    | December 2020   |

### HD75 Hearing Aid ANSI S3.22-2009 Measurements (2cc Coupler)

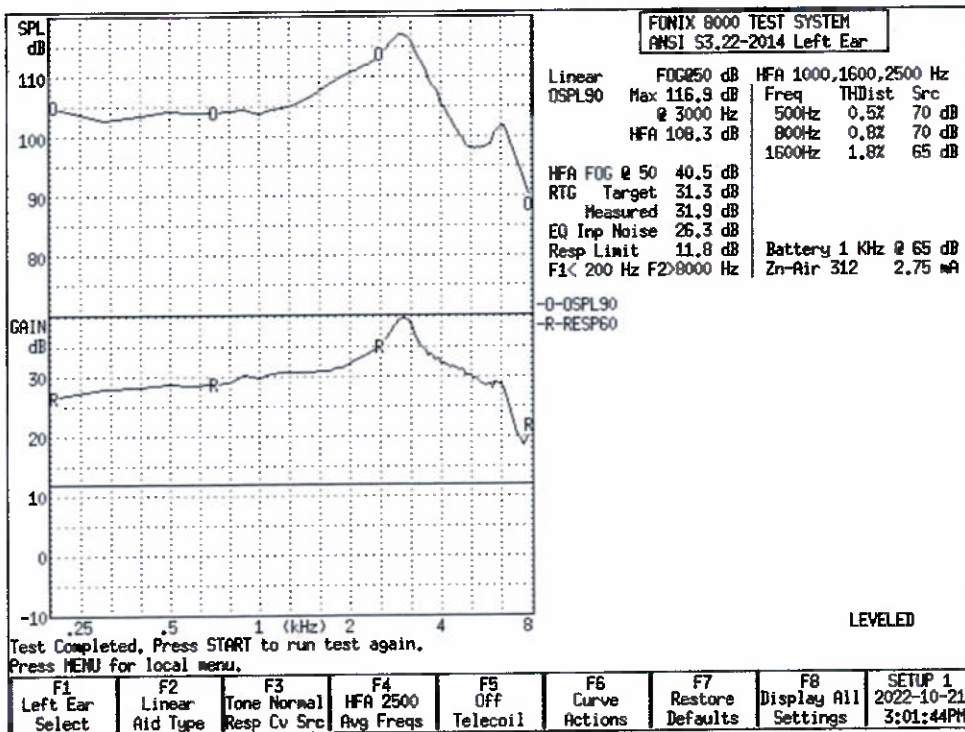

**WARNING: If you are younger than 18, do not use this.** You should go to a doctor, preferably an ear-nose-throat doctor (and ENT), because your condition needs specialized care. Over-the-counter hearing aids are only for users who are age 18 or older.